sv BRUKSANVISNING

INSTRUCTIONS FOR USE en

VIKTIGT

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IMPORTANT

Please read all instructions before use.
The product must only be used in accordance with these
instructions. Misuse may endanger the patient's safety.

These instructions for use is also available on the website of the
legal manufacturer www.effectummedical.com/imprint/

Manufacturer:
Effectum Medical AG
Kirchgasse 11
CH-4600 Olten
www.effectummedical.com

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Informations for use

INTENDED PURPOSE

ReLink Care is a breakaway connector for intravenous (IV) therapy use. ReLink Care separates into two self-sealable parts when an excessive tension is exerted on the IV line, thereby avoiding accidental dislodgement of the catheter, spillage of fluids, and patient injury. The parts are reconnectable after disinfection.

Indications for use

Continuous and intermittent IV infusion where the IV catheter is secured to the patient using film dressing or equivalent.

Contraindications

Do not use ReLink Care for:

- IV infusion using an infusion pump.
- IV infusion of blood, blood products or other body fluids.

Intended users

ReLink Care is only intended for use by healthcare professionals.

Patient target group

Patients weighing over 2 kg receiving gravity activated IV therapy except for specified contraindications.

Instructions for use

The following information describes how ReLink Care is to be used. Users must read the instructions for use carefully before using ReLink Care for the first time to fully understand all features and indications for use.

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User environment

ReLink Care is intended to be used in hospitals and other healthcare facilities where gravity activated IV therapy is performed.

Intended interaction with patient

External use only. Indirect contact with patient through infusion fluid.

Duration of use

ReLink Care is a single use device intended to be used for a maximum of 24 hours and then changed to a new one.

Caution and limitations during use

ReLink Care should be connected to the extension tube and not directly to the IV catheter.

ReLink Care should only be connected to luer lock connections.

ReLink Care should not be transferred from one gravity IV line to another.

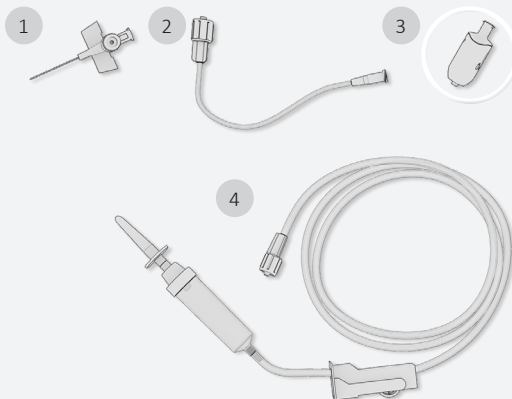
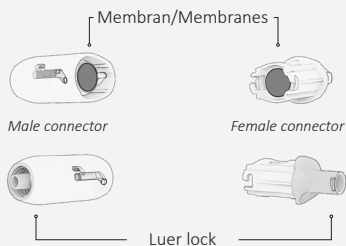
ReLink Care should not be left on the extension tube after the IV therapy session is finished.

ReLink Care must not be reconnected more than five times.

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

1. Venkateter
2. Kort förlängningsslang
3. ReLink Care
4. Gravitationsaggregat

ReLink Care shall be used together with an extension tube between 5 - 25 cm in length (*part 2 in system overview above*).

1. IV catheter
2. Short extension tube
3. ReLink Care
4. Gravity IV line

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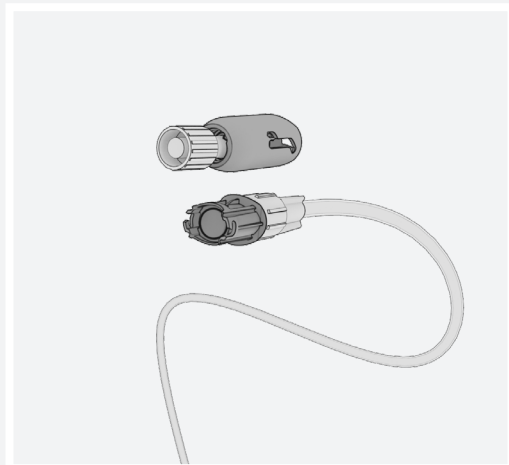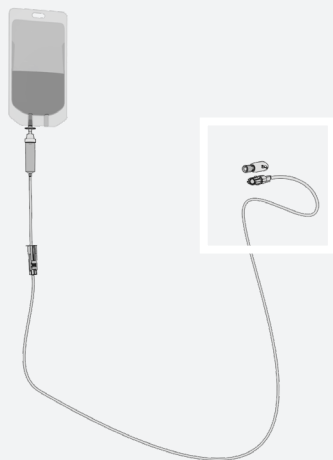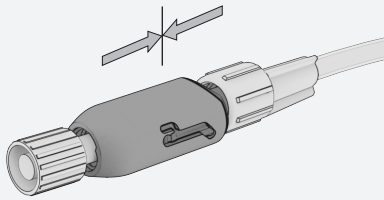
Make sure that the roller clamp is **closed**.

Unpack and handle ReLink Care using aseptic technique.

Remove the cap on female connector.

Connect the female connector to the gravity IV line.

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Align the two parts and push them together with a steady motion. Please note that when the parts are close to being fully connected, a final hard push is required to activate the membranes and open the flow path.

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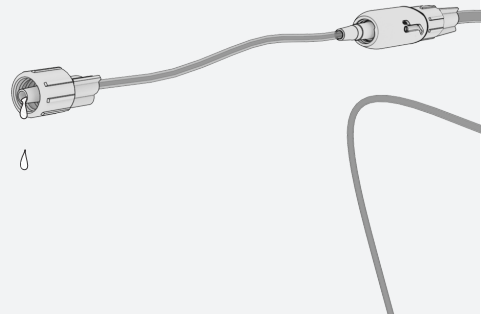
Remove the cap from the male connector and connect ReLink Care to the short extension tube (step 3). When ReLink Care is properly connected, the flow path is open, and fluid can pass through. This is controlled by priming (step 4) or by making sure the IV therapy can be started after step 5.

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
Prime the system.


If the system cannot be primed, it could be because ReLink Care is not properly connected. See step 2.

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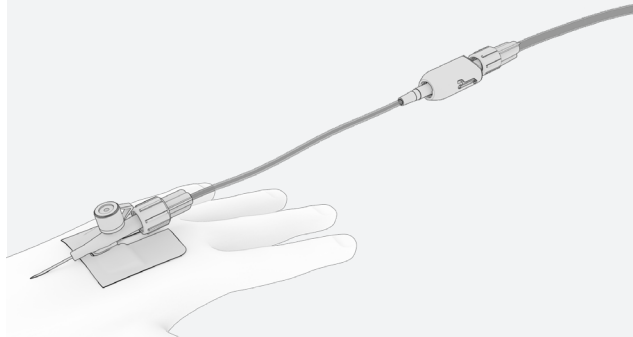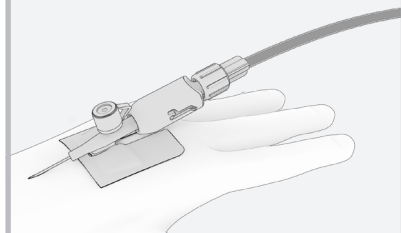
Connect the primed extension tube to the patient's IV catheter.

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ReLink Care **must never be connected directly** to the patient's IV catheter.

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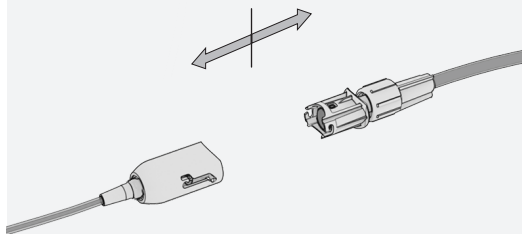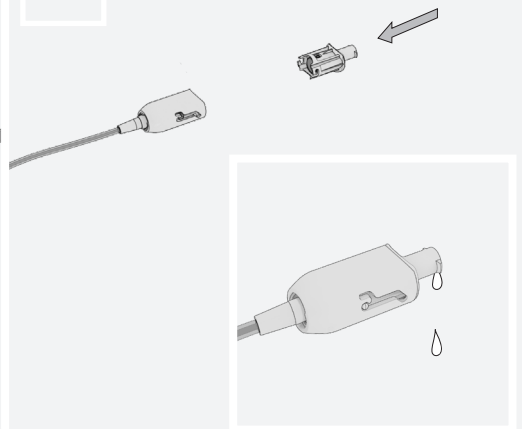
6b


If ReLink Care is accidentally disconnected (step 6a) while in use it can, after disinfection (step 7), be reconnected again. If required, one half of ReLink Care can be disinfected and the other one changed to a new one (step 6b).

If ReLink Care has already been reconnected five times and is again disconnected, the **entire ReLink Care must be replaced** with a new one.

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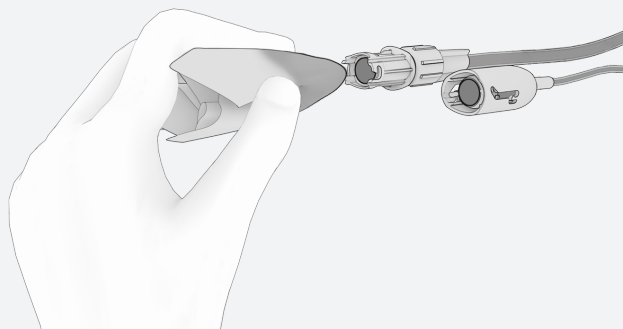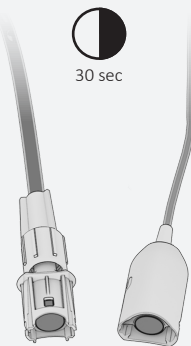
Check all surfaces of both ReLink Care parts. If any visible contaminants are present, replace both parts with a new primed ReLink Care according to the steps above (step 3). If no contaminants are visible, the parts can be disinfected.

Disinfect ReLink Care membranous parts by continuously rubbing the membrane surfaces for 15 seconds on each membrane with 70% isopropyl alcohol (IPA) wipes (step 7a). Create a pointed tip with the wipe to reach the membranes.

Allow a minimum of 30 seconds for the disinfection agent to dry (step 7b) before reconnecting the ReLink Care parts according to step 2 above.

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IMPORTANT

Dispose of used ReLink Care in a manner consistent with established biohazard procedures in accordance with local regulations.

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ReLink Care is made from Polycarbonate, ABS and Silicone and should not be used with fluids not compatible with these materials.

Flowrate of ReLink Drain is 230ml/min measured for viscosity of water, 1.0016 mPa-s at 20°C.

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Warnings

- Do not use ReLink Care with an infusion pump as this may cause leakage due to high pressure.
- Do not use ReLink Care for infusion therapy of blood, blood products or other body fluids as this may lead to blood agglutinates inside the device.
- Do not reuse ReLink Care as this may change its mechanical features and may cause device failure or bacterial infections.
- Do not connect ReLink Care directly to the patient's IV catheter as this may cause IV dislodgement.
- Do not use clamps or forceps to connect, tighten or disconnect ReLink Care from the gravity IV line or the extension tube, as this can lead to overtightening and damage to ReLink Care.
- Do not use the same ReLink Care for more than 24h as this may lead to leakage.
- ReLink Care should not be transferred from one gravity IV line to another as this may lead to cross contamination.
- ReLink Care should not be left on the extension tube after the IV therapy session is finished as this may cause leakage or backflow of blood from the patient in the device.
- Never connect any other devices to ReLink Cares membranous parts as this may lead to leakage or backflow of blood from the patient in the device.
- Do not use ReLink Care if the use-by date has passed or if the packaging or device is damaged as this may cause septic shock if the device is not sterile.
- Do not use more than two ReLink Care on the same patient per 24 hours as this may lead to toxic reaction from leachables.
- ReLink Care must not be reconnected more than five times as this may cause device failure causing leakage or organ dysfunction.
- ReLink Care must not be reconnected if not disinfected as this may lead to contaminants enter the bloodstream and cause sepsis.
- ReLink Care must not be attempted to be connected to any non-luer lock devices as this may endanger the patient's safety.

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All serious incidents that occur in relation to use of the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

Kontakt / Contact information

Effectum Medical AG | Kirchgasse 11 | CH-4600 Olten | info@effectummedical.com | www.effectummedical.com
AR / EC-REP: Interlinked AB | feedback@interlinked.care